

MEDICLEAN PLUS Disinfection and Detergent tablets for hospital use

MEDICLEAN *plus*



Presentation:

100 tablets in pots of White polyethylene.

Use Recommendations:

It is a product developed for use in hospitals, laboratories, dental centers, emergency rooms, dialysis centers and blood transfusion and other medical institutions in general.

Applications:

Mediclean Plus tablets were developed for use in disinfection and cleaning of surfaces with high risk of infection, in a quick, safe and precise way; allowing to clean and disinfect in one step and with just one product.

Shelflife:

3 years

Advantages of this presentation:

- ◆ Easily metered and stable tablets, with safe and precise concentrations.
- ◆ Allows fulfilling disinfection procedures, validating ppm of chlorine (mg/liter).
- ◆ Separate cleaning and disinfection is not required, for they are obtained with the created solution.

How to use MEDICLEAN Plus (Eg. Areas with high risk of infection)

1



ADD 3 TABLETS PER LITER OF WATER.

2



WAIT 5 MINUTES.

3



USE THE SOLUTION WITH A CLEAN CLOTH, RAG OR MOP OVER THE SURFACES THAT NEED DISINFECTION.